

Fionia^Vet

FioniaVet Inflatable Collar

Cat. No.	Size	Specifications Heavy Duty 600D Nylon Material			
		Outer Diameter	Inner Diameter	Neck Circumference	Weight
1091	XS	16 cm	4 cm	12 - 15 cm	100 g
1092	S	22 cm	6 cm	16 - 24 cm	150 g
1093	M	25 cm	10 cm	25 - 35 cm	200 g
1094	L	30 cm	15 cm	36 - 44 cm	250 g
1095	XL	36 cm	17 cm	45 - 55 cm	350 g
1096	XXL	44 cm	21 cm	56 - 65 cm	450 g

Instruction for Use:

1. Open the zipper and take out the air valve. Leave the air bladder in the cover.
2. To inflate the air bladder, either pinch the bottom of the air valve to inflate by mouth, or for better result, inflate by pump.
3. Do not overinflate and the air saturation should not exceed 90%. After inflating, cover tightly.
4. Press the air valve into the air bladder and zip up the cloth cover.
5. Insert your pet's everyday collar through the inner ring loops of the inflatable collar.
6. Stick the strap together for adjusting a perfect fit.
7. You can take out the air bladder and wash the cloth cover.
8. Open the lid of the air valve and pinch the bottom of the air valve, the air bladder will be deflated.

Washing Care:



 For Veterinary Use Only