

Smith+Nephew

Non-Medicated Paraffin Gauze Dressing

This information is intended for Healthcare Professionals only.



Description

Jelonet* is a soft paraffin dressing that is not medicated, making it ideal for use with topical antibiotics or antiseptics. It is soothing and low-adherent, and allows the wound to drain freely into an absorbent secondary dressing. **Jelonet** is available in a wide range of sizes, the largest of which is suitable for use in burns units.

Features

- Soft paraffin base
- Sterile leno weave presentation
- Comprehensive size range

Benefits

Soothes and protects the wound and allows free passage of viscous exudate

- **Jelonet** is soothing and low-adherent and allows the wound to drain freely into an absorbent secondary dressing.

Maintains shape, resists fraying

- **Jelonet** is a sterile paraffin tulle gras dressing made from open weave gauze. The gauze has interlocking threads which minimise fraying when the dressing is cut to shape.

Can be combined with topical medication of choice

Smith+Nephew

- **Jelonet** is not medicated and so is ideal for use with the topical antiseptic or antibiotic of choice.

Contraindications / Precautions

- Change dressing daily or more often according to the condition of the wound
- For leg ulcers, only use under medical advice
- Seek medical advice immediately for all serious wounds and burns, or if redness or discomfort occurs

Instructions for Use

Step 1:

Depending on presentation, open packet or peel down at chevron seal.

Step 2:

Remove dressing with forceps or gloved hand.

Step 3:

Put the dressing smoothly on the wound.

Step 4:

Use an appropriate absorbent secondary dressing to handle excess exudate if appropriate such as **Melolin** or **Melolite**.

Step 5:

Retain the dressing using a bandage or adhesive material, e.g. **Fixomull** or **Hypafix**.

<https://www.smith-nephew.com/australia/healthcare/products/product-types/tulle-gras-dressings/jelonet/>